3M
 St. Paul, Minnesota, USA
 1-800-243-4630
1860 Series Respirators

**THESE RESPIRATORS ARE APPROVED ONLY
 IN THE FOLLOWING CONFIGURATIONS:**

		RESPIRATOR COMPONENTS			CAUTIONS AND LIMITATIONS ²
		FILTERING FACEPIECE			
TC-	PROTECTION ¹	1860	1860S	8612F	
84A-0006	N95	X	X		ABCJMNP
84A-4526	N95			X	ABCJMNP

1. PROTECTION

N95 - Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply.

2. CAUTIONS AND LIMITATIONS

- A - Not for use in atmospheres containing less than 19.5 percent oxygen.
- B - Not for use in atmospheres immediately dangerous to life or health.
- C - Do not exceed maximum use concentrations established by regulatory standards.
- J - Failure to properly use and maintain this product could result in injury or death.
- M - All approved respirators shall be selected, fitted, used and maintained in accordance with MSHA, OSHA and other applicable regulations.
- N - never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- O - Refer to User's Instructions, and / or maintenance manuals for information on use and maintenance of these respirators.
- P - NIOSH does not evaluate respirators for use as surgical masks.

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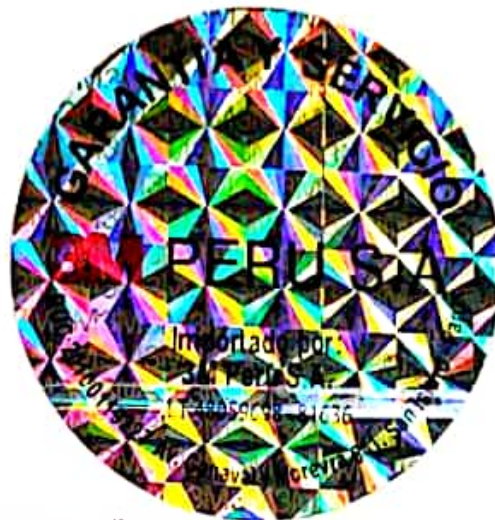
! **IMPORTANT:** Before use, the wearer must read and understand these user *Instructions* inside. **Misuse may result in sickness or death.**

Not intended for Individual resale.

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